TÜRKİYE TEK YETKİLİ DİSTRİBÜTÖRÜ

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FlexiOH® is the first of its kind- an orthopedic immobilizer that has the rigidity to hold the fractured part as well as it ensures proper skin ventilation. FlexiOH® comes as an easy to fix wearable material on the injured part. After wearing, light treatment completes the curing process and makes the FlexiOH® immobilizer rigid. This process is quick and only takes 3-10 minutes, depending on the size of FlexiOH® immobilizer. After the course of therapy, FlexiOH® can be removed easily without the use of any other external cast cutter/spreader. The design and material of FlexiOH® are engineered to provide superior comfort to the patient. A foam, which is in contact with the skin, is biocompatible (ISO 10993-10 certified) and provides cushioning against the hardened polymer. The FlexiOH® immobilizers are designed in such a way that the intermittent hollow spaces facilitate circulation of air and evaporation of sweat or water from the skin surface.

Radiolucency of FlexiOH® does not interfere with the radiological imaging and diagnosis.

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Solution / ÇÖZÜM

FlexiOH® provides these robust solutions for major issues related to conventional casts:

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Absolute washable orthopedic Immobilizer for immobilization of wrist joint and closed fractures at radius-ulna

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INTENDED USE

- Non-displaced / Stable / minimally displaced / Buckle fracture of distal radius fracture
- Carpal bone fracture other than Scaphoid / Trapezium
- As a removable splint during rehabilitation

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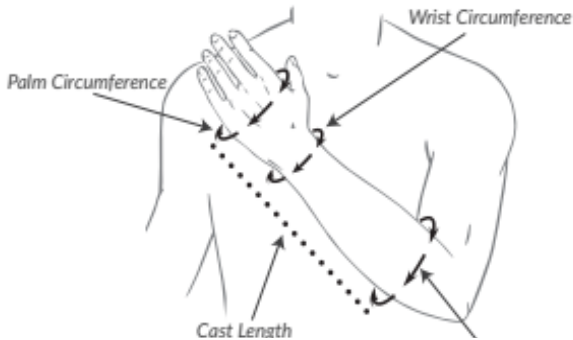
FlexiOH® short arm immobilizer comes in 5 different sizes: XS, S, M L and XL. All of the immobilizers vary in terms of palm circumference, wrist circumference, and arm circumference. For the best fit of FlexiOH® short arm immobilizer, taking the measurement of patient's hand is necessary before application

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MEASUREMENT GUIDE / ÖLÇÜM KILAVUZU

SIZE CHART / BEDEN TABLOSU

Size vs Measurement:	XS	S	M	L	XL
Palm Circumference:	13	16.5	18.5	21	23.5
Wrist Circumference:	11	14	16	18.5	21.5
Arm Circumference:	18	21	25.5	28.5	32
FlexiOH® Length:	24	28	30.5	34	38

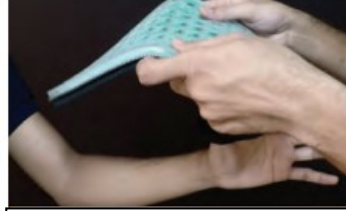
Notes :

- * Above sizes fit better as per dimensions mentioned in the table.
- * However FlexiOH® adapts to an oversize to a certain extent by stretchability.
- * We advise the healthcare professionals to choose size as per patient's comfort.

FlexiOH® short arm immobilizers are designed for distal radius/non-displaced/stable fractures or carpal bone fractures other than scaphoid fractures. This breathable, washable and light weight (~300 gms) immobilizer provides better comfort to the patients compared to conventional casts. For application of FlexiOH® short arm immobilizer, healthcare professional first need to apply it on the patient's hand with the zip and then cure it with the Rizycure® light device (recommended) or direct sunlight for around 3-10 minutes depending on the size of immobilizer. Once it is rigid, cut the puller of the zip with pliers so that patients can not remove the immobilizer by themselves.

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FlexiOH® provide 46% open area of skin to breathe, wash and exposure to sunlight which are necessary to maintain healthy skin.

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Absolute washable orthopedic Immobilizer for immobilization in the pre-operative, post-operative and rehabilitation period.

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INTENDED USE

- Acute swelling of fractured / injured limb / joint
- Pre-operative stabilization
- Post-operative support after surgery
- Ligament sprain / soft tissue injury
- Support during rehabilitation

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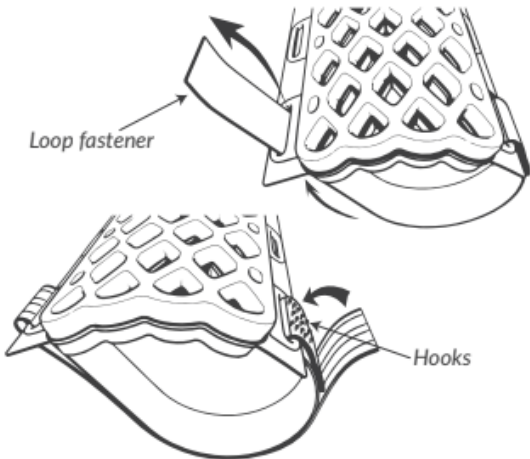
FlexiOH® Splints / Slabs are available in different sizes : 3 in X 10 in, 3 in X 12 in, 3 in X 15 in, 4 in X 12 in, 4 in X 15 in, 4 in X 18 in, 5 in X 12 in, 5 in X 15 in and 5 in X 18 in. All of the Splints / Slabs vary in terms of length and width.

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AVAILABLE SIZES

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3 in X 10 in	4 in X 12 in	5 in X 12 in
3 in X 12 in	4 in X 15 in	5 in X 15 in
3 in X 15 in	4 in X 18 in	5 in X 18 in

FlexiOH® Splints / Slabs are designed for immobilization in the pre-operative, post-operative, acute swelling and rehabilitation period. FlexiOH® has a built-in soft hook and loop fastener that secures the splint over the limb which is very comfortable for skin contact. With unique FlexiOH® technology, this splint gives better patient comfort along with breathability and washability. For application of FlexiOH® Splint / Slab, doctors need to apply it on the patient's limb with the soft hook and loop fastener and then cure it with the Rizycure® light device (recommended) for 3-10 minutes depending on the size of splint/slab. Once it is rigid, it will provide support during rehabilitation. With valcro, the splint can be removed / adjusted anytime for dressing and rehabilitation.

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FlexiOH® provide 30% open area of skin to breathe, wash and exposure to sunlight which are necessary to maintain healthy skin

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Absolute washable weight-bearing orthopedic Immobilizer for lower extremity fractures and soft tissue injury.

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INTENDED USE

- Lower leg fractures around ankle
- Fractures of foot bone
- Severe ankle sprains and strains or fractures.
- Immobilize leg or foot muscles and tendons in place after surgery to allow healing.

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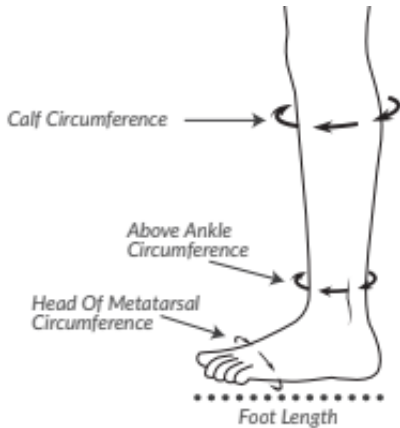
FlexiOH® Short Leg Immobilizer comes in 5 different sizes : XXS, XS, S, M and L. All of the immobilizers vary in terms of calf circumference, above ankle circumference, head of metatarsal circumference and foot length. For the best fit of FlexiOH® Short leg immobilizer, taking the measurement of the patient's leg is necessary before application.

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MEASUREMENT GUIDE / ÖLÇÜM KILAVUZU

SIZE CHART (in cm) / BOYUT TABLOSU (cm cinsinden)

Size vs Measurement:	XXS	XS	S	M	L
Calf Circumference:	22	24	29	34.5	40
Above Ankle Circumference:	17	20	21	24.5	26
Head of Metatarsal Circumference:	18	19.5	23	25.5	27
Foot Length:	16	20	22	24	27

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FlexiOH® Short leg immobilizers are designed as a weight bearing orthopedic casting technology for lower extremity fractures and soft tissue injury. It has a built-in zip system which makes the application process very easy and comfortable. With unique FlexiOH® Technology, this short leg immobilizer gives better patient comfort along with breathability and washability. For application of FlexiOH® short leg immobilizer, doctors need to apply it on the patient's leg with the zip and then cure it with the Rizycure® light device (recommended) or direct sunlight for 8-10 minutes depending on the size of immobilizer. Once it is rigid, it will take the shape of the patient's leg for support during immobilization.

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FlexiOH® provide 45% open area of skin to breathe, wash and exposure to sunlight which are necessary to maintain healthy skin

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FlexiOH®

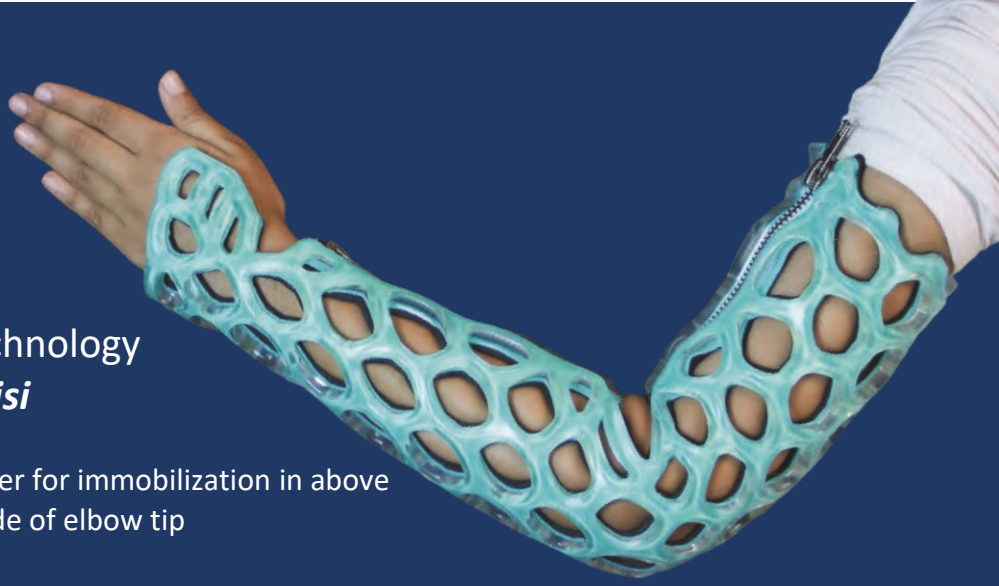
The NextGeneration

Orthopedic Immobilization Technology

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Absolute washable orthopedic Immobilizer for immobilization in above elbow, at the elbow knob and at the inside of elbow tip

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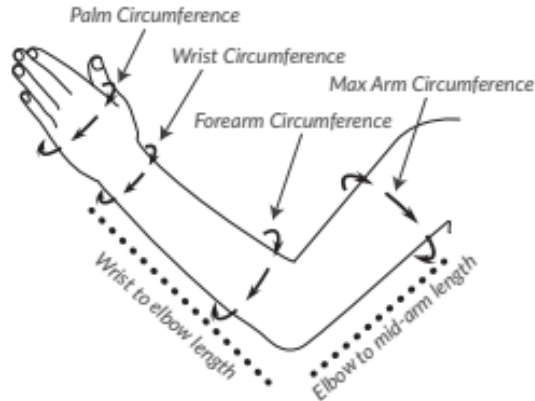
FlexiOH® Full Arm Immobilizer comes in 5 different sizes: XS, S, M, L and XL. All of the immobilizers vary in terms of palm, wrist, forearm circumference, maximum arm circumference, wrist to elbow length and elbow to upper arm length. For the best fit of FlexiOH® full arm immobilizer, taking the measurement of the patient's hand is necessary before application.

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MEASUREMENT GUIDE / ÖLÇÜM KILAVUZU

SIZE CHART (in cm) / BEDEN TABLOSU (cm cinsinden)

Size vs Measurement:	XS	S	M	L	XL
Palm Circumference:	13	16.5	18.5	21	23.5
Wrist Circumference:	12	14	16	18.5	21.5
Forearm Circumference:	17.5	21	25.5	28.5	32
Max. Arm Circumference:	16.5	24.5	29	34	40
Wrist to Elbow Length:	19.5	24	25.5	30	32.5
Elbow to Mid-arm Length:	15.5	19	23	26	29.5

Notes :

- * Above sizes fit better as per dimensions mentioned in the table.
- * However FlexiOH® adapts to an oversize to a certain extend by stretchability.
- * We advise the healthcare professionals to choose size as per patient's comfort.

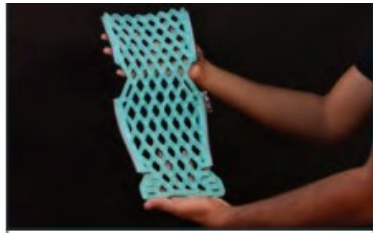
FlexiOH® Full arm immobilizers are designed for immobilization in above elbow (supracondylar), at the elbow knob and at the inside of elbow tip. The Full arm immobilizer is engineered with new features designed to provide secure and comfortable support. Patients can do daily activities while wearing this immobilizer. For application of FlexiOH® full arm immobilizer, doctors need to apply it on the patient's hand with the zip and then cure it with the Rizycure® light device (recommended) or direct sunlight for around 5-10 minutes depending on the size of immobilizer. Once it is rigid, it will take the shape of the patient's hand for support during the treatment.

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Put full arm immobilizer under the hand
Tam kol immobilizeri elin altına koyun

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FlexiOH® provide 35% open area of skin to breathe, wash and exposure to sunlight which are necessary to maintain healthy skin

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FlexiOH®

The NextGeneration

Orthopedic Immobilization Technology

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Absolute washable orthopedic Immobilizer for immobilization of thumb

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INTENDED USE

- Scaphoid fracture - seen or suspected
- De Quervain tenosynovitis / other ligamentous or soft tissue injuries to the thumb.
- Non-angulated, non-displaced, extra-articular fractures at the base of the first metacarpal.

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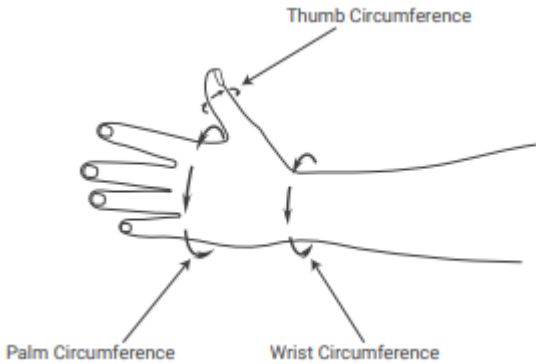
FlexiOH® Thumb Spica Immobilizer comes in 3 different sizes: S, M and L. All of the immobilizers vary in terms of palm circumference, wrist circumference, and thumb circumference. For the best fit of FlexiOH® Thumb Spica Immobilizer, taking the measurement of patient's hand is necessary before application.

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MEASUREMENT GUIDE / ÖLÇÜM KILAVUZU

SIZE CHART (in cm) / BOYUT TABLOSU (cm cinsinden)

Size vs Measurement:	S	M	L
Palm Circumference:	16	18	23
Wrist Circumference:	15	16	21
Thumb Circumference:	5.5	6.5	8.5

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FlexiOH® Thumb spica immobilizers are designed for immobilization of thumb. It has an inbuilt soft hook and loop fastener which is very comfortable for skin contact. It is used to provide support for thumb injuries (ligament instability, sprain or muscle strain), gamekeeper's thumb, de Quervain's syndrome or fractures of the scaphoid or first metacarpal. Doctors need to apply it on the patient's hand and then cure it with the Rizycure® light device (recommended) or direct sunlight for around 3-5 minutes depending on the size of immobilizer, until it gets rigid.

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JC OrthoHeal Pvt. Ltd.
OrthoHeal LLC